













Controls	Image	Lock	Unlock
Shark drive only			
Shark lights		<ol style="list-style-type: none"> 1. Press the on/off button for 4 seconds (while the chair is turned on) 	<ol style="list-style-type: none"> 1. Press the on/off button 2. Press the horn button twice (within 10 sec)
 Shark seats & lights			
DX 2 Rem 420			
DX 2 Rem 550			
			

Controls	Image	Lock	Unlock
 VR2 Drive			
VR2 Lights		<ol style="list-style-type: none"> 1. Press the on/off button until the system bleeps 2. Deflect the joystick forward until the system bleeps 3. Deflect the joystick in reverse until the system bleeps 4. Release the joystick the system will now make a long bleep 	<ol style="list-style-type: none"> 1. Press the on/off button To switch it on. 5. Deflect the joystick forward until the system bleeps 6. Deflect the joystick in reverse until the system bleeps 2. Release the joystick the system will now make a long bleep
VR2 seats & Lights			
R-net LCD			
R-net LED	